


RapidVet®-H (Feline A, B & AB) Instructions for Use

Dispense 1 drop of diluent into the Auto-Agglutination well. Dispense 1 drop of patient blood into the same well. Stir well for 10 seconds using a stirrer.


Auto-Agglutination Pre-Screen

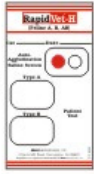

If no agglutination is observed in the well, then you may proceed with the test.


Dispense 1 drop of diluent into each remaining well.

Step 1:

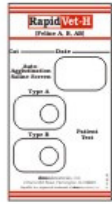


Dispense 1 drop of patient blood into the well marked Type A and dispense one drop of patient blood into the well marked Type B.

Step 2:

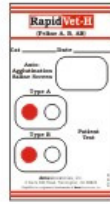


Stir Type A well with a stirrer for 10 seconds. Using a NEW stirrer, stir the Type B well for 10 seconds.

Step 3:

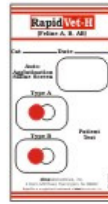

Add a second drop of diluent to the well marked Type A. Do not stir the well again with a stirrer.

Step 4:

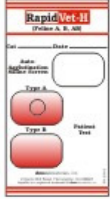

Pick the card up and rock and tilt by wrist action for 1 minute, or until agglutination occurs in one of the wells, whichever is sooner.

Step 5:



Read the results:

Step 6: See Photo Interpretation Guide

Position the card at a 10° - 20° angle to dry.

Step 7:



RapidVet is a registered trademark of **dml**laboratories, inc., 2 Darts Mill Road, Flemington, NJ 08822 Tel.: (908) 782-3353 or (800) 567-4367 in U.S. and Canada; Fax: (908) 782-0832